

# HOSPITAL BEDS

## Setup Instructions

(Manual, full electric, semi-electric and bariatric)

- **MANUAL:** Use the crank provided to raise and lower the head and foot sections, as well as the entire bed.
- **SEMI-ELECTRIC:** There are 4 buttons (located on the pendant). The top 2 raise and lower the head of the bed, the bottom 2 raise and lower the foot of the bed. Use the crank provided to raise and lower the entire bed.
- **FULL ELECTRIC:** There are 6 buttons (located on the pendant). The top 2 raise and lower the head of the bed. The bottoms 2 raise and lower the foot of the bed. The middle 2 raises and lowers the entire bed.

## Cleaning Instructions

The mattress can be cleaned with soap and water as needed. Keeping the mattress clean will help prevent infection.

## Operating Instructions

1. Always make sure that the two locking wheels (located at each end of the bed) are currently locked.
2. Only push one button at a time and only operate one section of the bed at a time. Pushing two buttons at a time can damage the bed or stop that function from operating.
3. The bed is provided with side rails. To raise the rail, pull up on each end of the rail until it locks into place. To lower the rail, pull out the button of the rail (located on the bottom side of the rail) with one hand and use the other hand to slowly ease the rail down.
4. Make sure to check and/or tighten rails on a weekly basis for patient safety.
5. Never place any items under the bed.
6. Do not attempt to take the bed apart.

### **\*\* Bed Rail-Related Entrapment Safety Information \*\***

Both split and full rails have the potential to cause fall-related injuries as well as entrapment. Actions to prevent deaths and injuries from entrapment include reducing excessive gaps and openings.

Mattresses provided by Enos Home Oxygen are appropriately sized for the bed frames.

The use of non-standard mattresses is forbidden because of the risk of gaps and entrapment that could occur when the mattress is pushed to one side of the bed

## Things You Can Do If Your Electric or Semi-Electric Bed Fails

1. Make sure that the bed is **PLUGGED IN**.
2. Make sure that the wall outlet the bed is plugged into **HAS POWER**.
3. If the bed still fails to operate, use the emergency hand crank to position the bed and **CALL ENOS HOME OXYGEN** for service.